




Short-term patient stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.

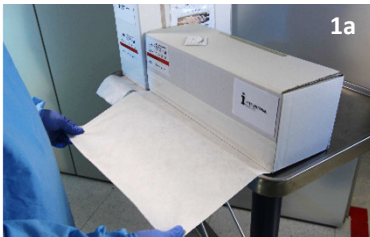

1a




1b

1

Estimate the required length of iFIX Fleece and cut.



2a

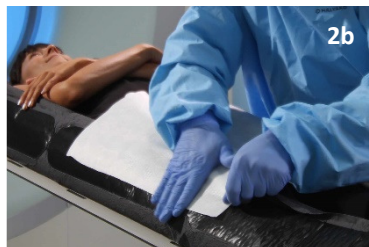

2b


2

On the opposite side of the table attach the iFIX Fleece to anchor product, using the textured surface. Using the wrong surface of the fleece may lead to patient injury.

**Anchor products:** iFIX Patch, iFIX Patch Adapter and iFIX Slot Adapter



3a

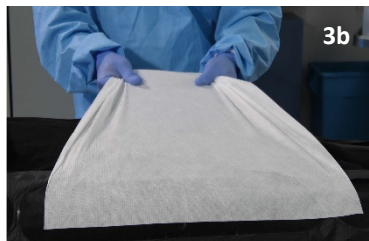

3b

3

Pull iFIX Fleece tightly, up and over the patient.

Do not apply the iFIX Fleece too tight, or it can restrict the blood circulation and the patient's breathing.



4a




4b

4

Adhere the iFIX Fleece to the anchor product on the other side of the table.

If necessary, adjust iFIX Fleece to increase or decrease stabilization. Confirm the stability of the setup before conducting the procedure and readjust if necessary. Forgetting it may lead to patient injury.

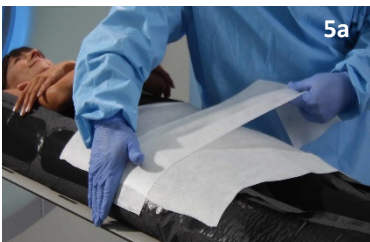

5a



5b

5

Repeat this procedure with a second piece of iFIX Fleece if desired.

See Instructions for Use for respective anchor products.

After use, dispose the fleece. Reusing an iFIX Fleece may lead to patient injury.



**CONSULT PRODUCTS HERE**

Scan the QR-Code on the left to consult the different available Fleece products

## WARNINGS AND PRECAUTIONS

- iFIX is indicated for medical procedures during which the patient's tissue is required to maintain a specific position over a short period of time.
- The system may only be used by medical professionals.
- Non-Sterile
- Hypoallergenic
- Single patient use; Do not reuse
- Not made with natural rubber latex
- Safe for Use in MRI



[www.interventional-systems.com](http://www.interventional-systems.com)

**MANUFACTURER:**  
iSYS Medizintechnik GmbH  
Bergwerksweg 21  
6370 Kitzbühel, Austria  
Tel: +49 (8232) 99 41 956  
info@interventional-systems.com

**INTERVENTIONAL-SYSTEMS** is a registered trademark  
**iSYS** is a registered trademark  
**iFIX** is a registered trademark  
**iFIX** technology patent pending  
© 2018 iSYS Medizintechnik GmbH

