



Short-term patient stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.



1 Select appropriate iFIX Head Mask size for the patient.



2 Place mask over patient face ensuring eyes, nose, and mouth are exposed. Do not apply the iFIX Head Mask too tight, as it can restrict the blood circulation and the patient's breathing.



3 Attach iFIX Head Mask tabs to anchor product (red arrow). Confirm the stability of the setup before conducting the procedure and readjust if necessary. Forgetting it may lead to patient injury.

Anchor products:
iFIX Patch, iFIX Patch Adapter



4 If necessary adjust iFIX Head Mask to increase or decrease stabilization. See Instructions for Use for respective anchor products.

After use, dispose the head mask. Reusing an iFIX Head Mask may lead to patient injury.

CONSULT PRODUCTS HERE

Scan the QR-Code below to consult the different available Head Masks



WARNINGS AND PRECAUTIONS

- iFIX is indicated for medical procedures during which the patient's tissue is required to maintain a specific position over a short period of time.
- The system may only be used by medical professionals.
- Non-Sterile
- Not made with natural rubber latex; Hypoallergenic
- Safe for Use in MRI
- Single patient use; Do not reuse